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Instruction Manual

DDtempMED

DDtempMED

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Material-related characteristics

Composition [wt.%]

Polymethylmethacrylat (PMMA)	99 - 99,8
Coloring pigments	0,2 - 1

Physical specifications

Flexural strength	[MPa]	78 - 90
Flexural modulus	[MPa]	2300 - 2988
Water absorption	[$\mu\text{g}/\text{mm}^3$]	19 - 25
Residual monomer content	[%]	< 2

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R_x only Caution: U.S. Federal law restricts this device to sale by or on the order of a dentist.

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Instruction Manual

- Indications**
DD temp MED are pre-colored dental milling blanks out of PMMA for the manufacture of temporary crowns and bridges for long-term application in the oral cavity up to 12 months.
- Contraindications**
 - intolerance to the contained components
 - bridges with two molar pontics
 - too little connector cross section
 - model casting with clasps, bars, narrow sublingual bars and transversal connectors
- Safety information**
Please pay attention to the information in the instruction manual in its current version. Avoid the inhalation of dust particles during processing. Wear protection gloves, safety glasses and a dust mask to prevent irritations of eyes, skin and respiratory system.
- Handling and storage**
Only use clean, defect-free material out of undamaged packaging and make sure that the blanks are always stored in their original container in a dry and cool place, protected from light. Avoid contact with fluids.

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Instruction Manual

5. Designing

Handling this medical device should only be allowed to trained staff.

When designing the objects, the minimum wall thickness should not be less than 0.8 mm in the cervical area and 1.5 mm in the occlusal area. The connection cross sections between crowns and pontics of a construction must not be less than 12 mm². In general all constructions should be designed in a way to enable the compensation and the transfer of the mastication forces to the abutment teeth.

6. Milling and further processing

Only use one-edged milling tools in the milling machine or wet grinding procedures!

Further processing:

Tungsten carbide burs like the one you use for soft reline materials are suitable for rough adaption. For the finalization common cross-toothed millers can be used. Polishing should be carried out with regard to a low contact pressure, otherwise the resulting heat might lead to smearing or melting.

7. Cleaning

- ultrasonic bath (max. 40 °C) with mild cleaning agent

Do not use the following methods for cleaning:

- steam blasting
- autoclave method
- organic solvents and acids (e.g. acetone, hydrochloric acid)
- highly concentrated alcohols
- chemical denture cleaners
- abrasive cleaning agents

8. Mounting

Bonding extra-orally:

Choose primer suitable for the adhesive. Only use special adhesives, adapted to the material you want to combine with the PMMA. Pay attention to the instructions of the adhesive manufacturer.

Cementation:

For temporary cementations are recommended temporary cements or glass ionomer cements

9. Disposal

Remains of PMMA blanks can be disposed of in the regular household garbage.

We reserve the right to make changes because of the continuous development of our products. Please find the current version of the instruction manual on our website: www.dentaldirekt.de/en.

This version replaces all previous versions.