## DE DEUTSCH

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MD EU Medizinprodukt

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MD EU Medical device

#### Product description:

DD Uni Bond is a one-component silanisation agent for durable adhesive bonding between indirect restorations (also zirconium dioxide) and luting composites, as well as for the pretreatment of glass fibre reinforced composite posts.

#### Indications:

DD Uni Bond is used as a coupling agent to achieve an optimal bond strength between indirect restorations made of either silicate ceramics, aluminium oxide, zirconium dioxide, metal or composite, as well as glass fibre-reinforced composite posts, on the one side and methacrylate-based luting composites on the other side.

#### Contraindications:

DD Uni Bond contains methacrylated phosphoric acid ester and 3-methacryloxypropyltrimethoxysilane. DD Uni Bond should therefore not be used in patients with a known hypersensitivity (allergy) to these ingredients.

#### Patient target group:

DD Uni Bond is suitable for use in all patients without any age or gender restrictions.

#### Performance features:

The product's performance features satisfy the requirements of the intended use and the relevant product standards.

#### User:

DD Uni Bond should only be used by a professionally trained dental practitioner.

#### Use:

##### Preparation:

Bring material to room temperature before use.

##### Luting of indirect restorations:

Surface conditioning of indirect restorations, including those made from zirconium dioxide, creates optimum conditions for longterm bonding of restoration and luting composite.

- Once the fit of the restoration has been checked, prepare its bonding surfaces according to the manufacturer's instructions for use by etching or sandblasting, rinse carefully with water and dry with oil-free air.

##### When using the bottle:

Dispense DD Uni Bond onto a DD Bio Z applicator or a disposable brush. Alternatively, dispense DD Uni Bond onto a mixing pallet. Close the bottle after use.

- Apply DD Uni Bond onto the prepared surfaces with a DD Bio Z applicator or disposable brush, allow to take effect for 60 s and dry carefully with oilfree air.
- Apply the luting composite as described in the respective instructions for use and place the restoration.

##### Luting of glass fibre reinforced posts:

- After appropriate preparation of the root canal and position check of the glass fibre reinforced post, shorten the post to operational length according to the manufacturer's instructions for use, then clean with alcohol and dry with oil-free air.
- Coat the entire post with DD Uni Bond (for details on dispensing from bottle: see above), allow to take effect for 60 s and dry carefully with oil-free air.
- Cement the post in the root canal according to the instructions of the manufacturer.

##### Warnings, precautionary measures:

- Contamination of the conditioned bonding surface (e.g. saliva or blood) may reduce the bonding strength of DD Uni Bond. Therefore, repeat the surface conditioning steps as described above.
- Contact of DD Uni Bond with skin/mucosa or eyes may have an irritating effect and should be avoided.
- Our information and/or advice do not relieve you of the obligation of checking that the products supplied by us are suitable for their intended purpose.

##### Constituents (in descending order according to content):

Acetone, 3-methacryloxypropyltrimethoxysilan, methacrylated phosphoric acid ester

##### Storage:

Storage at 4°C-23°C. Refrigeration is recommended. Do not use after the expiry date.

##### Disposal:

Dispose of the product in accordance with local regulations.

##### Reporting obligation:

Serious events such as death, temporary or permanent serious deterioration of a patient's, user's or other person's health condition and a serious risk to public health that arise or could have arisen in association with the use of DD Uni Bond must be reported to the manufacturer and the responsible authority.

##### Note:

The Summary of Safety and Clinical Performance of DD Uni Bond can be found in the European database on medical devices (EUDAMED – <https://ec.europa.eu/tools/eudamed>).

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