DDcontrast® firing paste

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Properties:

DD contrast® firing paste – the ultimate Ceramic Firing Paste – Peg Putty

- Conveniently stabilizes ceramic restorations on firing pegs.
- Zero clean-up is required – just a blast of compressed air is sufficient.
- No tiresome divesting by way of scraping out and sand blasting.
- No drying time is required.

Directions for use:

Apply a small amount from the syringe directly into the crown/coping or on the back of the veneer or inlay. Please make sure that the material does not touch the visible surfaces of the ceramic. The DD contrast® firing paste must not be regarded as a stump replacement material, but serves purely as an aid for placement on the firing tray.

Fix and adjust your object on the holding pins of the firing tray or place it directly on the tray. No drying time is required. The last step is to start your firing process as usually, DD contrast® firing paste does not require any adjustments of the firing cycle. After firing simply clean off DD contrast® firing paste with a purposeful stroke of compressed air. Please do not reuse the DD contrast® firing paste once fired because the material contracts a bit and a perfect fit during further firing cycles cannot be guaranteed.

DD contrast® firing paste contains small fibres that might clog the syringe tip – please simply use an instrument to unplug the tip again.

The sealing cap of the syringe has to be put on immediately after usage in order to prevent a dry out of the material!